


POLAR EQUINE FT1 HEALTHCHECK

Getting Started Guide

POLAR[®]
LISTENS TO YOUR BODY

1. PRODUCT PARTS

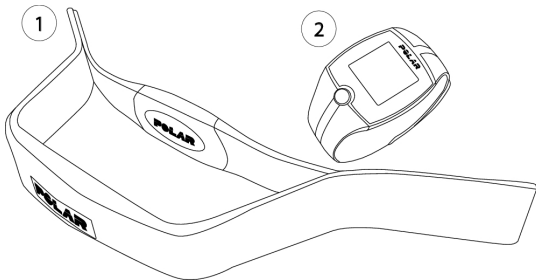



1. **T31 transmitter with a handlebar:** the transmitter sends the ECG accurate heart rate signal to the training computer.
2. **FT1 training computer:** the training computer displays the heart rate.



User manual in German, French, Italian, Spanish, Finnish and Swedish can be downloaded at www.polar.com/support.

For video tutorials, go to http://www.polar.com/en/polar_community/videos.

2. USING FT1 HEALTHCHECK

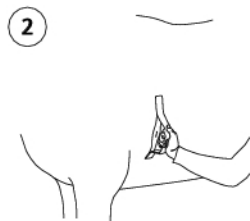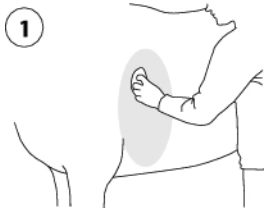

1. Wet the horse's hair abundantly as indicated (1). Alternatively, you can use electrode gel.
2. Place the transmitter against the hair of the horse (2). Keep the FT1 training computer within 1 meter / 3 feet from the transmitter.
3. Press the button to start. Your horse's heart rate appears in 5 seconds.
4. Press the button to stop.

3. TROUBLESHOOTING

Abnormal Heart Rate Readings

- Make sure that the transmitter is firmly against the horse.
- Make sure that the horse's hair under the transmitter is thoroughly moistened.
- Accumulated sweat and dirt may interfere with the measurement of the signal from the heart to the transmitter. Check if the transmitter needs cleaning.
- If the heart rate measurement still does not work, the battery of your transmitter or training computer may be empty. Have the battery checked by authorized Polar service personnel. Contact your distributor/dealer for details.



For a video tutorial on using FT1 Healthcheck, go to http://www.polar.com/en/polar_community/videos.

4. IMPORTANT INFORMATION

Care and Maintenance

Follow the caring instructions to ensure reliable measurement and to maximize the life span of the transmitter and training computer. The following instructions will help you fulfill the guarantee obligations.

Keep the transmitter and training computer in a cool and dry place. Do not store in a damp environment, in non-breathable material (a plastic bag or a sports bag) or with conductive material (a wet towel). Do not expose to direct sunlight for extended periods. Clean with a mild soap and water solution, dry with towel. Never use alcohol or any abrasive material (steel wool or cleaning chemicals). Do not bend or stretch

the transmitter. This may damage the electrodes. Do not press the button of your training computer under running water.

Service

During the 12-month guarantee/warranty period we recommend that you have service done by an authorized Polar Service Center only. The warranty does not cover damage or consequential damage caused by service not authorized by Polar Electro. For contact information and all Polar Service Center addresses, visit www.polar.fi/support and country specific websites.

Guarantee

Limited International Polar Guarantee for Equine Products

- Polar Equine products are developed and manufactured for the usage in equine sports. Each product endures rigorous testing before it leaves the factory.
- This guarantee does not affect the customer's statutory rights under applicable national or state laws in force, or the customer's rights against the dealer arising from their sales/purchase contract.
- This limited Polar international guarantee is issued by Polar Electro Inc. for customers who have purchased this product in the USA or Canada. This limited Polar international guarantee is issued by Polar Electro Oy for customers who have purchased this product in other countries.
- Polar Electro Oy/Polar Electro Inc. guarantees the original customer/purchaser of this device that the product will be free from defects in material or workmanship for twelve (12) months from the date of purchase.
- The receipt of the original purchase is your proof of purchase!
- The guarantee does not cover the battery, normal wear and tear, damage due to misuse, abuse, accidents or non-compliance with the precautions; improper maintenance, cracked, broken or scratched cases/displays, modified or altered products or their parts and electrode belt and strap of sensors.
- The guarantee does not cover any damage/s, losses, costs or expenses, direct, indirect or incidental, consequential or special, arising out of, or related to the product.
- Items purchased second hand are not covered by warranty, unless otherwise stipulated by local law.

- During the guarantee period, the product will be either repaired or replaced at any of the authorized Polar Service Centers regardless of the country of purchase.

Guarantee with respect to any product will be limited to countries where the product has been initially marketed.

Regulatory information is available at www.polar.com/support.



This marking shows that the product is protected against electric shocks.



This crossed out wheeled bin marking shows that Polar products are electronic devices and are in the scope of Directive 2002/96/EC of the European Parliament and of the Council on waste electrical and electronic equipment (WEEE) and batteries and accumulators used in products are in the scope of Directive 2006/66/EC of the European Parliament and of the Council of 6 September 2006 on batteries and accumulators and waste batteries and accumulators. These products and batteries/accumulators inside Polar products should thus be disposed of separately in EU countries.

Copyright © 2012 Polar Electro Oy,
FI-90440 KEMPELE.

Polar Electro Oy is a ISO 9001:2008 certified company.

All rights reserved. No part of this manual may be used or reproduced in any form or by any means without a prior written permission of Polar Electro Oy. The names and logos marked with a ® symbol in this user manual or in the package of this product are registered trademarks of Polar Electro Oy.

Disclaimer

Please note that these guarantee terms shall be applied to Polar Equine products instead of any other Polar Electro guarantee term, even if other terms may appear in some product documentation.

Manufactured by

Polar Electro Oy

Professorintie 5

FIN-90440 KEMPELE

Tel +358 8 5202 100

Fax +358 8 5202 300

www.polar.fi

POLAR[®]
LISTENS TO YOUR BODY